

# PREVENT THE FIRST SIGNS OF AGEING WITHOUT SURGERY

## What is Clear + Brilliant® ?

Clear + Brilliant® is a gentle laser resurfacing treatment which offers 2 wavelengths : The Original (1440 nm) and the Perméa (1927 nm).

## How does Clear + Brilliant® work ?

Clear + Brilliant uses fractional laser technology to create hundreds of thousands of microscopic treatment zones in the upper layers of the skin, which replaces damaged skin with healthy looking tissue and yields younger-looking skin.\*\*1

## What Can Clear + Brilliant® Do for You?

If you're battling the effects ageing can have on your skin, or trying to address it before the battle even starts, Clear + Brilliant can help. With this gentle laser treatment, you can help prevent the visible signs of ageing and address the overall dulling effects time and the environment can have on your skin.<sup>3,6</sup>

## Treatment Results:

Though results vary, patients frequently report younger looking skin and a radiant glow<sup>1</sup> after treatment. Clinical studies highlights include:

- Improvement in the look of fine lines<sup>1</sup>
- Reduction in the appearance of pores<sup>2</sup>
- Improvement to skin tone and texture<sup>2,3</sup>
- Improvement in skin's overall appearance<sup>1,2</sup>

## Why opting for Clear + Brilliant routine:

- Non-surgical and non-invasive
- Quick and gentle treatment<sup>4</sup>
- Little to no social downtime<sup>5</sup>
- Glowing youthful looking skin<sup>1,2</sup>
- Suitable for all skin types and tones<sup>2</sup>

clear + brilliant®

[www.clearandbrilliant.co.uk](http://www.clearandbrilliant.co.uk)

### INDICATION

Clear + Brilliant Laser System (1440nm and 1927nm handpieces) is indicated for dermatological procedures requiring the coagulation of soft tissue and general skin resurfacing.

### IMPORTANT SAFETY INFORMATION

- The following contraindications are routine for many laser treatments and may also be associated with non-ablative Clear + Brilliant Laser System treatments. You may not be an appropriate candidate for a treatment if you:
  - Have been diagnosed with or possibly have actinic keratosis, melasma, rosacea, or other significant skin conditions (e.g. skin cancer, active infections, cold sores, open wounds, rashes, burns, inflammation, eczema, psoriasis);
  - Are predisposed to keloid formation or excessive scarring;
  - Have a condition that may compromise the immune system such as HIV, lupus, scleroderma, or systemic infections;
  - Are on systemic steroids (e.g. prednisone, dexamethasone), which should be rigorously avoided prior to and throughout the course of treatment;
  - Have a known sensitivity to light or if you are taking photosensitizing agents or medications;
  - Are using retinoids, which should be avoided for at least 2 weeks prior to and during treatment;
  - Are undergoing accutane™ treatment or taking drugs in a similar class;
  - Have skin that is still recovering from a cosmetic procedure, such as a chemical or mechanical peel, or laser resurfacing; and
  - Have had botox injections, or dermal fillers within the past 2 weeks.

### REFERENCES

1. CSR\_CB (1440nm) Clinical Report\_2011.pdf; 2. CB\_Saedi\_JAAD\_Evaluating facial pore and skin texture after low-energy nonablative fractional 14401; 3. ARTICLE\_CB (1927nm)\_Folder\_PRIMEJOURNAL\_2013.pdf; 4. REF-CBT-0026-C+B treatment times; 5. ARTICLE\_CB (1927nm)\_Folder\_PRIMEJOURNAL\_05Jun13\_Submitted and Accepted\_201; 6. REF-CBT-0034-K120433\_C and B\_Histology Study Report\_Permea; 7. CB IFU.pdf. 8. Brauer JA, Alabdulrazzaq H, Bae YS, Geronemus RG. J Drugs Dermatol. 2015;14(11):1262-1267.

\*salesdata. \*\*Clear + Brilliant Laser System Operator Manual.

Please read Instructions for Use (IFU) for more information regarding the medical device. Clear and Brilliant is a medical device (CE 0197)

®/™ are trademarks of Bausch Health Companies Inc. or its affiliates.

CLB.0014.EU.22-CB

clear + brilliant®

Pioneer in laser therapy for the prevention of the early signs of ageing



Shannon  
Model and  
Actual Patient



# A CLEAR STORY OF BRILLIANT SUCCESS

- ▶ A leader in preventative anti-ageing laser treatments since 2011
- ▶ Good patient satisfaction level<sup>3</sup>
- ▶ Over 20 published clinical studies
- ▶ +2.5 million procedures worldwide\*
- ▶ Good safety profile
- ▶ 2 synergistic treatments in 1 system:
  - Original handpiece offers a targeted penetration for a deeper treatment\*\* improving skin tone and texture and preventing the early signs of ageing.
  - Perméa handpiece offers a broader penetration<sup>8</sup> for an improved skin's overall appearance.



# RESULTS THAT MATTER

Clear+Brilliant® treats fine lines, the appearance of pores, skin tone and texture<sup>1,3</sup>



Before Clear + Brilliant®



9 weeks after 2 treatments



Before Clear + Brilliant®



1 week after 6 treatments



Before Clear + Brilliant®



1 week after 6 treatments

Photographs courtesy of Solta Medical Aesthetic Centre.  
Individual results may vary. Unretouched pictures.

# WHAT TO EXPECT

## Pre-treatment consultation

- ▶ All Clear + Brilliant laser treatments begin with a consultation with your healthcare professional to see if Clear + Brilliant is right for you .
- ▶ Your healthcare professional will discuss the best Clear + Brilliant treatment plan with you.

## A Typical Clear + Brilliant Treatment:

- ▶ Requires about 30 minutes for a complete session<sup>4</sup>
- ▶ May involve application of topical numbing cream to your face prior to treatment in order to provide additional comfort
- ▶ Is delivered by your healthcare professional, who will gently guide the Clear + Brilliant handpiece across the target area and actively treat your skin for approximately 30 minutes.<sup>4</sup>
- ▶ Utilises the patented Intelligent Optical Tracking® technology to ensure your healthcare professional maintains proper contact with your skin and provides a uniform application to all treated areas<sup>7</sup>
- ▶ With Clear + Brilliant Perméa® treatments, a topical antioxidant serum might be applied to the skin immediately after treatment.

